

XD 3.6.10 Release Notes

Contents

1	Introduction	3
1.1	Purpose.....	3
1.2	Scope	3
1.3	Alternative formats	3
1.4	Disclaimer of Warranty.....	3
1.5	Definitions	3
2	XD 3.6 Release Notes	4
2.1	Known Limitations	4
2.1.1	FUXD-4527 Snapshots of empty views replace previous snapshots	4
2.1.2	FUXD-4499 Statistics tables can be truncated in report	4
2.1.3	FUXD-4386 Report templates not available to all users	4
2.1.4	FUXD-3565, FUXD-3294 Asian characters are not properly rendered in an exported PDF Report	4
2.1.5	FUXD-3116 Error attempting to export resampled Enhanced CT.....	4
2.1.6	FUXD-3115 Out of memory on exporting Enhanced CT	4
2.1.7	FUXD-2666 Encoding Holes in RT Structure Sets	5
2.1.8	FUXD-2645 Large Secondary Captures may slow down load times	5
2.1.9	FUXD-2557 Inconsistent region shadowing when data bindings are changed.....	5
2.1.10	FUXD-2537 Inconsistent Regions across a deformable registration.....	5
2.1.11	FUXD-2535 Slice range export can crop the image being exported	5
2.1.12	FUXD-2533 MIP Generation displayed when playing Cine with MIP view present.....	5
2.1.13	FUXD-2515 Region Statistics Obscured by Cine Slider.....	6
2.1.14	FUXD-2499 Transforming Regions Across Deformable Registrations	6
2.1.15	FUXD-2488 Exporting Gated PET Data	6
2.1.16	FUXD-2340 Paint Tool usage on off-axis views	6
2.1.17	FUXD-2312 Launching XD with no supported data.....	6
2.1.18	FUXD-2045 Goto Ruler created on rotated view	6
2.1.19	FUXD-1379 - Some avi files do not play in VLC	6
2.1.20	FUXD-1165 – No overwrite prompt on saving images to file from the Image Gallery ...	6
2.1.21	FUXD-618 - Fused MIPs Degrades Performance When Visible During Registrations.....	7
2.1.22	RT-85 – Missing XD icons	7
2.1.23	RT-2 – Paint/Erase tools with small brush sizes do nothing in some views.....	7
2.2	Product Notes.....	7
2.2.1	Create Ruler From Region Tool	7
2.2.2	Region Intensity Distribution Graph with multiple datasets	7
2.2.3	CT Window and level presets ‘abdomen/pelvis’ and ‘chest’ presets have same value .	7
2.2.4	XD automatically adjusts out of range window and level values	8
2.2.5	Calculation of SUV.....	8
2.2.6	Registration of Hybrid Data with inconsistent frame of Reference.....	9
2.3	Thin Client Deployment.....	9
2.3.1	FUXD-2669 – Active text control and cine can skip slices.....	9
2.3.2	FUXD-2654 - XD Appears unresponsive via Thin Client	9
2.3.3	Quantization of Colors on Client Systems.....	10

2.3.4	Creating a Viewer Application Over Network is not recommended	10
2.3.5	First Connecting to the Server may take longer the first time	10
2.3.6	Shared Configuration Items	10
2.3.7	Saving to the File System – Default Location	10
2.3.8	Fast Layout Switcher does not work as Intended	10
3	Device Details	11

DRAFT

XD 3.6.10 Release Notes

1 Introduction

1.1 Purpose

The purpose of these Release Notes is to:

- Communicate any known limitations for Mirada’s XD 3.6.10 release.
- Provide information on certain product features.

1.2 Scope

The scope of these Release Notes is Mirada’s XD 3.6.10 product

1.3 Alternative formats

An online version of the XD 3.6.10 instructions for use (including these release notes) is available at <https://mirada-medical.com/eifu-xd-3-6-10/> and may contain updates that were made in the time since you were delivered your copy of XD 3.6.10.

A physical copy of the XD 3.6.10 instructions for use (including these release notes) is also available upon request from support@mirada-medical.com.

1.4 Disclaimer of Warranty

Mirada Medical makes no representations or warranties, either expressed or implied, by or with respect to anything in this document, and shall not be liable for any implied warranties of merchantability or fitness for a particular purpose or for any indirect, special or consequential damages.

**Copyright © 2019, Mirada Medical.
All rights reserved.**

References to other companies and their products use trademarks owned by the respective companies and are for reference purpose only.

1.5 Definitions

Term	Definition
AC	Attenuation Corrected
DICOM	Digital Images and Communication in Medicine.
NAC	Non Attenuation Corrected
PACS	Picture Archiving and Communication System
ROI	Region of Interest
SCP	Service Class provider (DICOM node to receive DICOM data)

2 XD 3.6 Release Notes

2.1 Known Limitations

2.1.1 FUXD-4527 Snapshots of empty views replace previous snapshots

When using the **Take snapshot of views** tool, if a view is selected that does not contain data or is out of the field of view, the previous snapshot taken is replaced.

2.1.2 FUXD-4499 Statistics tables can be truncated in report

When a large number of statistics are calculated, such as for numerous findings, the statistics table can be truncated when added to a report page. The data can still be included in the report using text fields and the **Copy Table** button of the Findings Table.

2.1.3 FUXD-4386 Report templates not available to all users

When saving report templates, saving the template for all users does not make the template available to other users.

2.1.4 FUXD-3565, FUXD-3294 Asian characters are not properly rendered in an exported PDF Report

The Report feature within XD permits the user to export a document as PDF.

Asian characters (including Chinese, Japanese and Korean) do not render properly into the PDF document, even if they are displayed correctly within XD.

There is no workaround to this issue until XD ships with support for CJK fonts. Text in reports should be constrained to a different language – for example, English – if the PDF does not support the native language.

2.1.5 FUXD-3116 Error attempting to export resampled Enhanced CT

XD does not support DICOM export of Enhanced CT as a resampled volume. Any attempt to perform a resampled export of Enhanced CT data will cause this error to be reported in the Messages window:

```
Uncaught exception on event dispatch thread (IllegalStateException: null)
```

There is no workaround to this issue until XD supports the resample export of Enhanced CT data.

2.1.6 FUXD-3115 Out of memory on exporting Enhanced CT

XD may report an out of memory error when exporting Enhanced CT as original data.

There is no workaround to this issue within XD. The Enhanced CT data loaded into XD must be transferred to the destination by some other means.

2.1.7 FUXD-2666 Encoding Holes in RT Structure Sets

XD encodes each hole within a region as a separate contour when writing to a DICOM RT Structure Set. This should be noted when reviewing XD generated contours on a RT Planning system.

2.1.8 FUXD-2645 Large Secondary Captures may slow down load times

XD allows the viewing of secondary capture images. These can include slice ranges, which can potentially be very large and affect load times and the amount of available memory. If performance is not as expected then these secondary captures should be de-selected before launching XD.

2.1.9 FUXD-2557 Inconsistent region shadowing when data bindings are changed

For PET/CT gated data, changing the data bindings after region creation may appear to exhibit inconsistent shadowing behavior. For threshold regions, the original shadow remains when the data binding is changed in the Data Management screen. When the threshold is edited the shadow appears on the new bound data, but is not removed from the old one as it should be (nor changed on the old one).

However when this region is converted to a painted region, using the editing brush, the old shadow disappears.

To work around this issue, it is recommended that any changes that need to be made to the bindings in Data Management screen are performed before creating regions.

2.1.10 FUXD-2537 Inconsistent Regions across a deformable registration

A region created on the overlay layer of a fused view may not visually match an identical region created on the same dataset in a non-fused view if a deformable registration is in place between the datasets visible in the fused view.

This is a visual effect only and does not affect the underlying region statistics.

2.1.11 FUXD-2535 Slice range export can crop the image being exported

When exporting slice ranges, if the width of the view being exported exceeds the height, the image will appear slightly cropped in the preview displayed when exporting the slice range. This can be avoided by adjusting the width of the exported image (default 300) to be more than the height until the image preview contains all the desired image. An alternative is to slightly zoom out the view required before selecting the option to export the slice range.

2.1.12 FUXD-2533 MIP Generation displayed when playing Cine with MIP view present

When playing a cine for gated data with a MIP view on screen, a MIP generation message is displayed each time a different phase is displayed causing slow playback of the MIP. This can be avoided by selecting a layout that does not contain a MIP view.

2.1.13 FUXD-2515 Region Statistics Obscured by Cine Slider

The within-view VOI statistics may be slightly obscured if the cine slider is active in the image views. To view the obscured text, disable the cine slider (navigation tools). A hot key can be set to do this in [Tools->Options].

2.1.14 FUXD-2499 Transforming Regions Across Deformable Registrations

Regions transformed across deformable registration may not correspond exactly to the transformation indicated by the cross-hairs at all points on the boundary. This is a visual effect only and does not affect the underlying region statistics.

2.1.15 FUXD-2488 Exporting Gated PET Data

Exporting an original or re-sampled phase from a Gated PET series results in the exported data containing the gating related meta-information. In some systems this may result in the data appearing to be an entire gated series, whereas it is only one volume.

2.1.16 FUXD-2340 Paint Tool usage on off-axis views

When using the region paint and eraser tool, the depth can be set using the depth slider control. This depth is calculated from an axis oriented frame. When the paint tool is used in an rotated (off-axis) view, the depth applied to the painting operation may not be as expected. To avoid this issue, check the results after painting and adjust the depth as necessary when painting in off-axis views.

2.1.17 FUXD-2312 Launching XD with no supported data

If XD is started from the Mirada Application Launcher and no supported data is found, XD will close without displaying any message. If XD does not appear as you expect, check that at least one supported dataset is selected.

2.1.18 FUXD-2045 Goto Ruler created on rotated view

If a ruler is created on a rotated (off axis) view after which the orientation is changed, when the Goto ruler option is selected, the ruler will not be displayed. To workaround this issue, take a bookmark after creating the ruler. Double clicking the bookmark will then display the ruler.

2.1.19 FUXD-1379 - Some avi files do not play in VLC

Some avi files created in Mirada XD will not play in all media players, such as VLC. They will, however, play correctly in Windows Media Player and Apple's Quicktime.

2.1.20 FUXD-1165 – No overwrite prompt on saving images to file from the Image Gallery

When saving Images from the Image Gallery to the file system, no warning is displayed should the folder being written to contain files with the same name as the system generated file names.

It is recommended that each time images are saved to file from the Image gallery, they are written to a new folder.

2.1.21 FUXD-618 - Fused MIPs Degrades Performance When Visible During Registrations

If a fused MIP is visible when registrations are being computed then each completed registration is applied to the Fusion MIP which takes a number of seconds to complete. This is most noticeable when performing manual registrations.

It is recommended that fused MIP pop up window views are closed prior to performing registrations and that a layout without a fused MIP view is selected before performing registrations.

2.1.22 RT-85 – Missing XD icons

If XD is installed alongside the Mirada RTx product and RTx is subsequently un-installed, the XD icons in Application Launcher will not be present. To restore the XD icons, run repair from the XD application entry in Windows Control panel/un-install a program. Note: this will need to be run from a user account with the appropriate permissions to install applications.

2.1.23 RT-2 – Paint/Erase tools with small brush sizes do nothing in some views

When painting or erasing a region with the paintbrush size set to be smaller than a voxel, no voxels will be painted. This issue can be avoided by increasing the paintbrush size.

2.2 Product Notes

2.2.1 Create Ruler From Region Tool

The create ruler from region tool creates a bi-ruler (2D ruler) by calculating the long and short axes from the region selected. This bi-ruler is always taken from the smooth representation of the region. If the region display has been set to display voxel mask based regions, then the generated bi-ruler will appear to not exactly match the region.

2.2.2 Region Intensity Distribution Graph with multiple datasets

When multiple datasets are loaded into a particular role (e.g. gated CT, Multiphase CT) and a region has been propagated from one of the datasets to the others, then the Region Intensity Distribution Statistics will be displayed in the image views and the statistics displayed will pertain to the dataset for which they are displayed. The Intensity Distribution graph cannot be viewed in this case as no individual region can be selected. In addition, the statistics on the Intensity Distribution tab on the Findings Table will not be available.

2.2.3 CT Window and level presets 'abdomen/pelvis' and 'chest' presets have same value

Switching between 'abdomen/pelvis' and 'chest' window and level presets for CT datasets will appear to make no difference to the image visualisation.

No workaround is required. For the current release of the software the CT window and level presets for these two body parts have been intentionally set to the same value.

2.2.4 XD automatically adjusts out of range window and level values

If the user enters custom values to define the window & level by SUV, and the maximum value entered by the user is greater than twice the maximum SUV in the dataset, Mirada XD will automatically adjust the values to provide a suitable range.

No workaround is required as this is intended behaviour. Users should try to enter a suitable range of values based on the range of SUV found in the dataset.

2.2.5 Calculation of SUV

When using SUV calculation during PET assessment certain assumptions are made with regard to the reference time for the acquisition of the data series. Variability in interpretation of requirements outlined in the DICOM Standard with regard to determination of the start reference time during acquisition and the time of tracer injection may result in variability in the SUV values calculated by different vendors.

It is important to note that due to inconsistency of approach throughout the industry, the acquisition time used in SUV calculation may be any of the acquisition times presented in the DICOM data. It is equally important to note that SUV is affected by a number of physiological factors which cause variability (ref Von Schulthess, Clinical Molecular Anatomic Imaging, Lippincott, Williams and Wilkins, 2003 (ISBN: 0-7817-4144-0), pp. 86-87). Taking these two factors into account, SUV can be thought of as a simplified measure of radiopharmaceutical uptake which has a complementary rather than directive role in the assessment, treatment and staging of disease.

Note also that SUV calculations will only be possible for datasets that have all appropriate information. This includes:

- data source must be a PET Emission;
- activity must be expressed in appropriate units (BQML);
- appropriate decay correction must have been applied;
- half-life for radiopharmaceutical must be known.

Should SUV calculation not be possible, the SUV items on the Quantification Settings dialogs (accessible clicking on the modality overlays or by selecting Quantification Settings from the Image menu) will be marked as Unavailable.

Versions of the DICOM Standard prior to 2006 specify only a Radiopharmaceutical Start Time for PET series. This is the actual time of radiopharmaceutical administration to the patient for imaging purposes. However, the standard did not specify a date corresponding to this event, assuming it is the same date as the acquisition date. In some cases with long half-life radiopharmaceuticals, this may not be the case and there would be a need to specify a different date should the radionuclide creation date be different from the acquisition date. However, this situation could not be handled previous versions of the DICOM Standard. Since 2006 the DICOM Standard has been amended to explicitly include the date as well as the time for the radiopharmaceutical administration by introducing the Radiopharmaceutical Start Date

Time tag. However, not all vendors have implemented the change and there remains a source of ambiguity regarding the administration date.

Due to the omission of this information in older versions of the standard, some vendors express the actual date and time of the radiopharmaceutical administration in proprietary tags. Should the Radiopharmaceutical Start Date Time not be present, Mirada XD will check for the presence of the proprietary tag. Should the proprietary tag also be missing then Mirada XD will need to make an assumption about the date.

Mirada XD makes an initial assumption that the date corresponding to the Radiopharmaceutical Start Time is the same as the Acquisition Date. This assumption works well, except when the acquisition has been performed over midnight. In such cases, the initial assumption will result in Acquisition Date/Times which appear to precede the assumed Radiopharmaceutical Start Date/Time. If Mirada XD finds that the Acquisition Date/Times appear to precede the assumed Radiopharmaceutical Start Date/Time, it adjusts the assumed Radiopharmaceutical Start Date/Time by subtracting 24 hours. This correction will work correctly for datasets for which dates and times cross midnight, but not in other cases as there is not enough data in the DICOM header to assume any other option.

In summary, Mirada XD will use the Radiopharmaceutical Start Date/Time information in the data if present (through the standard tag or proprietary tags) and calculate it as specified above if *only* the time is present.

2.2.6 Registration of Hybrid Data with inconsistent frame of Reference

Some PET/CT scanners are known to create PET and CT data with inconsistent frames of reference. XD will preserve the frame of reference for this type of data (i.e. not register the PET to the CT) as long as the data matches the pre-configured values for this behaviour defined in XD. Please contact Mirada support for further information or to add additional scanners to the system configuration.

2.3 Thin Client Deployment

This section is only applicable for users that access through the Mirada supplied thin client installation.

2.3.1 FUXD-2669 – Active text control and cine can skip slices

Using the active text control or the cine play control on the thin client deployment can cause XD not to display every slice. It is recommended that these navigation methods are not used if it is required that every slice is displayed.

2.3.2 FUXD-2654 - XD Appears unresponsive via Thin Client

Occasionally, when running via thin client, XD will seem to be unresponsive or seem extremely slow to update to user interaction.

If this situation occurs, press the “Restore” button in the top right of the XD window and then “Maximize” and then the application will continue to respond as normal.

2.3.3 Quantization of Colors on Client Systems

The system is configured to use 32-bit color on connecting to the server. This setting must not be changed. Grayscale images displayed on the client are identical to those displayed when XD is installed locally. However, the user should note that there is a minor difference in the displayed images of a remote client and a local XD when using non-grayscale color maps. This effect only affects the displayed image and there is no difference in any of the quantification values. It is recommended that images are also reviewed using grayscale colormaps.

2.3.4 Creating a Viewer Application Over Network is not recommended

Creating a viewer to a drive over a network connection (e.g. the clients hard drive) takes substantially longer than creating to the local server on the file system.

To workaroud this issue, it is recommended that the viewer is created to the local server file system and copied via the operating system across the network to the desired location.

2.3.5 First Connecting to the Server may take longer the first time

On connecting to the Mirada Application Launcher running on the server for the first time, it may take up to two minutes to logon while Windows configures the users profile.

2.3.6 Shared Configuration Items

If any configuration items (e.g. report templates, layouts) are created by a user, then they will not be available to the other clients until the next time they connect.

In addition to this, Report Templates added for all users will not be available for selection on the report screen until they have been set as the default template for the review mode. This can be set in the Review Modes tab within [Tools>Options].

Shared layouts will be picked up by the system and displayed in the Review Modes tab also. They will not be available for selection until they are made available for a particular review mode.

2.3.7 Saving to the File System – Default Location

If any items are saved from XD or the Application Launcher, the default save location is in 'My Documents', i.e. C:\Users. This location is on the server, not on the client machine. As long as the network shares and permissions have been configured by IT, it is possible to save to the clients machine directly.

2.3.8 Fast Layout Switcher does not work as Intended

The CTRL+TAB method of switching layouts does not display the selector, it simply selects the next layout. It is recommended that this method of selecting layouts is not used for this deployment.

3 Device Details



In compliance with Council Directive 93/42/EEC.



Manufactured by:

Mirada Medical Ltd.
New Barclay House
234 Botley Road
Oxford
OX2 0HP
United Kingdom



Date of manufacture: March 2019



<https://mirada-medical.com/eifu-xd-3-6-10/>